




HUMAN CARE

Your Life. Your Way.



Mobi Pro Flexi

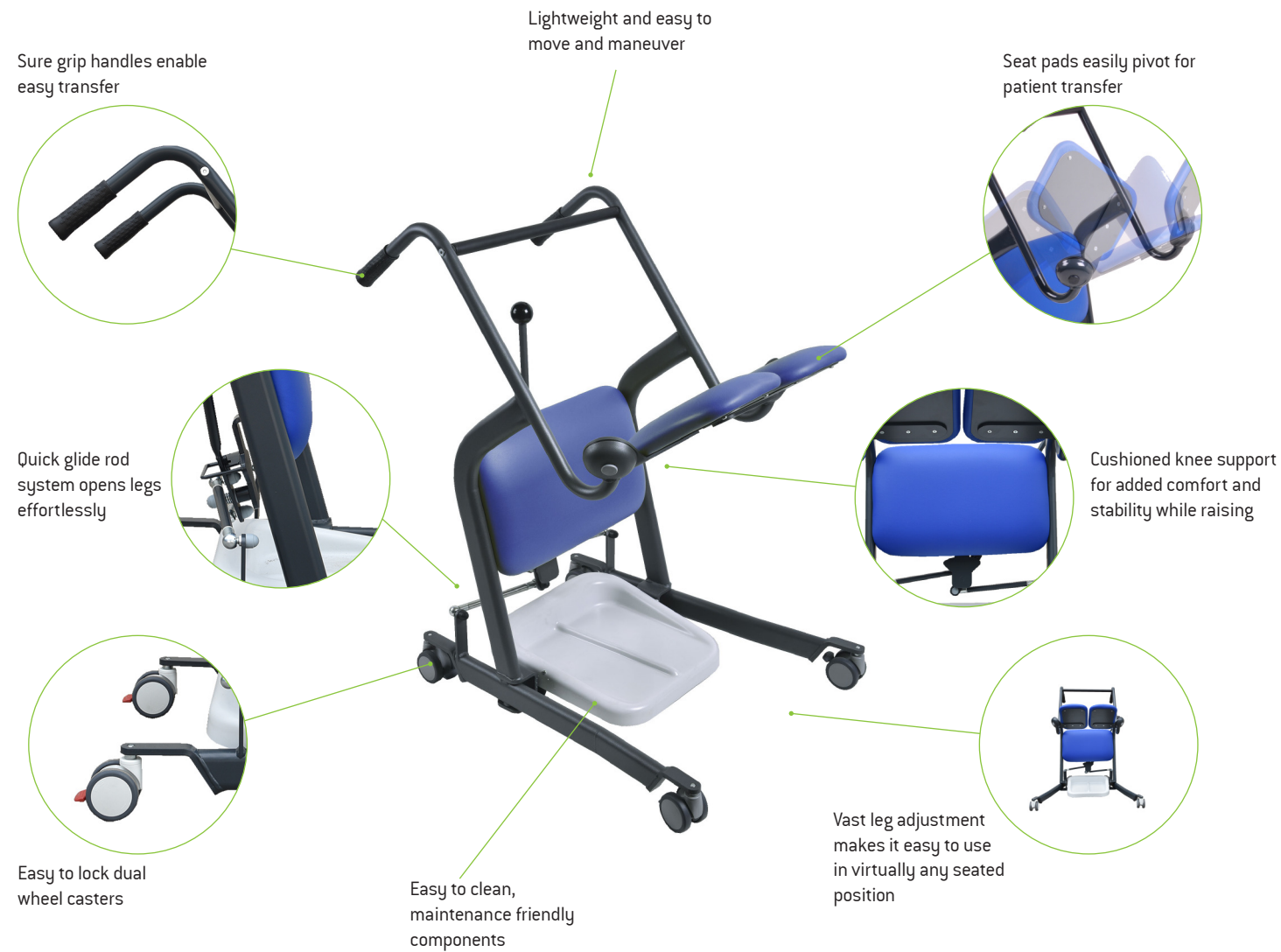


The Mobi Pro Flexi


A versatile and safe option for patient mobilization

The Mobi Pro Flexi is an ergonomic and compact mobile assistive mobility device ideal for people with low leg strength and some trunk balance who experience difficulty walking. With its rotating padded seats and knee support, the Mobi Pro Flexi is an excellent substitute for a wheelchair and its small turning radius makes it ideal for smaller spaces. The Mobi Pro Flexi is equipped with a mechanical mechanism to spread the legs, ensuring it will work with most seating. The vast adjustment range is easily set to three settings and it can safely be used in a moisture area such as a toilet. The Mobi Pro Flexi features rear caster brakes for added security and is virtually maintenance free.

Patient Suitability

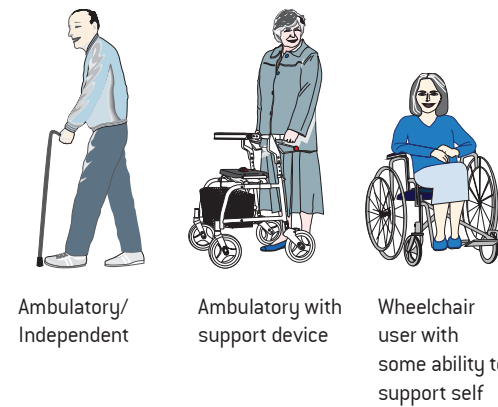

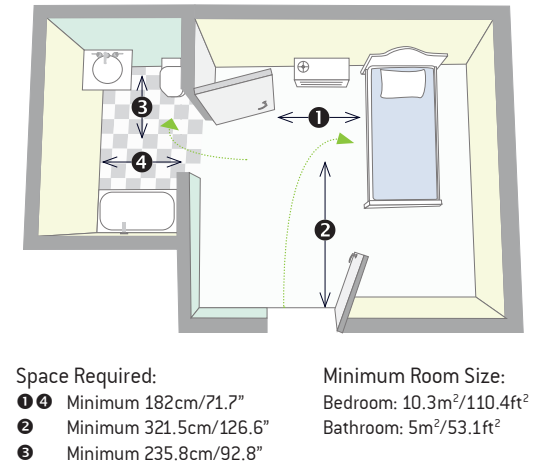


Environment Suitability



Part no.	Width	Length	Height	Wheel Diameter	Adjustable Leg (Min-Max)	Handle Height (Inclined at an Angle)	Wheel Size	Product Weight	Weight Capacity
92500	58.5cm/23"	97cm/38.2"	104cm/41"	7.5cm/3"	43cm - 70cm/17" - 27.5"	92cm - 100cm/36.2" - 39.4"	7.5cm/2.95"	19.6kg/ 43.2lb	150kg/330.6 lb

Safe, Efficient and Easy Transfers

Step 1	Step 2	Step 3
 <p>With the seats rotated upwards, bring the Mobi Pro Flexi towards the patient. The legs may either be open or closed depending on the transfer. Engage the brake.</p>	 <p>The patient rests their feet on the footplate and firmly grasps the handles to raise themselves.</p>	 <p>Once the patient is standing, swivel the seat pads into position so that the patient may sit down. Release the brake and proceed with the transfer.</p>

- Encourages patient mobility and independence
- Easy transfer to and from a toilet area
- Accommodates most seating types
- Allows eye contact between caregiver and patient
- Minimizes manual handling by caregiver thus increasing caregiver safety



AUSTRALIA

Unit 9, 271 Wells Road
Chelsea Heights, Victoria 3196
Phone:- +61 3 8773 1111
Fax:- +61 3 9773 4008

Email: info.au@humancaregroup.com

CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone:- 613.723.6734
Fax:- 613.723.1058

Email: info.ca@humancaregroup.com

NETHERLANDS

Elspeterweg 124
8076 PA Vierhouten
Phone:- + 31 577 412 171
Fax:- +31 577 412 170

Email: info.nl@humancaregroup.com

UNITED STATES

8006 Cameron Street, Suite K
Austin, TX 78754
Phone:- 512.476.7199
Fax:- 512.476.7190

Email: info.us@humancaregroup.com

SWEDEN

Corporate Office

Årstaängsvägen 21B
117 43 Stockholm
Phone:- +46 8 665 35 00
Fax:- +46 8 665 35 10

Email: info.se@humancaregroup.com



HUMAN CARE
Your Life. Your Way.