Petermann Badehilfe Peter Pan® PM IV Lang
Petermann Bathlift Peter Pan® PM IV Long

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1 Allgemeine Informationen

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Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

The Bathlift Peter Pan® Long is a mechanical aid and serves to support the patient while lowering and raising them in the bathtub and during the Bathing process.

It is used specifically of patients in home care as well as other health care facilities.

A Bathlift is used by people with restrictions, to enter and leave the bathtub or to stand up and lay down independently or need support during the bath procedure. By the use of the Bathlift Peter Pan Long the patient can be bathed, washed and cared.

The use of Bathlift Peter Pan® Long should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The product may only be used for the specified purpose and under the specified conditions.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Item no.: 42322-L Bathlift Peter Pan Long®

1.2 Accessories

Item no.: EZM-4628 handle

Item no.: EZM-4639-04 cover for reclining surface

Item no.: EZM-4655 set of suction feet

1.3 Scope of delivery

Bathlift Peter Pan® Long – basic device

Seat back

Cover for reclining surface

Handle

Box for handle

User manual

1.4 Storage, transport

Store/transport at -5°C – 40°C
Other: protect from sunlight

1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **ten years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of cleaning and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- It is essential to ensure that the number of users is sufficient to ensure a safe transfer and the bathing process in the given situation, also in regard to the medical condition of the patient.
- Ensure that the bathing process is performed by the users in a back-friendly manner.
- Weight limits must be observed (135 kg)
- Ensure that the seat and backrest can move freely during use
- Ensure that the feet and hands as well as the user's free hand are clear during lowering or raising of the Bathlift Peter Pan® Long.

2.1 Symbols used

	Wipe off		max. load capacity see item 6
	Do not bleach		Manufacturer
	Do not tumble		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label

2.2 Function check

The Petermann Bathlift Peter Pan Long® is designed with a stainless steel frame and therefore doesn't require any special maintenance. A visual inspection for defects is required before each use. The device may not be used if there are any noticeable defects.

Periodic inspections should be performed at least once per year; i.e. visual inspection of the load-bearing parts (function test). For sanitary reasons, the device must be cleaned regularly.

➤ **Defective products may no longer be used!**

3 Using the product

3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since Bathlift Peter Pan® Long is suitable for all applications involving people who require assistance.

This product may be reused.

The frequency of further use depends on the respective condition of the device. On the basis of the MPG (German Medical Devices Act) "Processing of Medical Devices" § 4, para. 2 MPBetreibV (Ordinance on Operators of Medical Devices).

A safety inspection of the device must be performed before it is used again.
Please request our dealer brochure "Reuse/processing of bathlifts"

If you don't use the Bathlift Peter Pan® Long for an extended period, store the Bathlift Peter Pan® Long in the uppermost position so that the spring mechanism is unloaded.

3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (see item 2), the product may not be used if the patient has any of the following conditions:

- Patients, who are not allowed to bathe
- Patients with elevated muscle tone (e. g. spastic tetraplegia)
- Patients with reduced muscle tone (e. g. ALS or MS)

3.3 Application risks

- Improper side loading → Risk of tipping
- Improper use of the handle → Risk of entrapment
- Backrest is not inserted correctly - Backrest tilts when loaded → Risk of falling

3.4 Areas of application

The product can be used in home care as well as other health care facilities.

3.5 Application illustration

N/A

3.6 Application

3.6.1 General description

The Petermann Bathlift Peter Pan® Long is used in home care as well as other health care facilities. It was designed exclusively for use in a bathtub.

3.6.2 Setup in the tub

- The Petermann Bathlift Peter Pan® Long is fully assembled and ready for immediate use.
- Place the Bathlift Peter Pan® Long **beside** the bathtub
- Place the included crank in the crank receiver in the front on the Bathlift Peter Pan® Long seat
- Turn the crank in the counter clockwise direction until the device is in the uppermost position
- Place the cover on the seat
- **Important:** Insert the top into the cover first
- Insert the sides into the cover and secure the snap fastener at the bottom
- Place the Bathlift Peter Pan® Long as far back as possible in the bathtub. The rear feet of the Bathlift Peter Pan® Long should rest on the floor of the bathtub just before the place where the tub curves up.
- Press the seat down firmly so that the two front suction cups attach securely.
- Then the Bathlift Peter Pan® Long is positioned securely in the tub.

3.6.3 Operation

- The Bathlift Peter Pan® Long is operated with the hand crank.
- The Bathlift Peter Pan® Long can be stopped at any height.
- The hand crank can be removed at any bathing height.
- The hand crank can be placed in the crank box.
- Turn to the left → Bathlift Peter Pan® Long is lowered
- Turn to the right → Bathlift Peter Pan® Long is raised
- Maximum number of turns → 57 (from the highest to the lowest position)

3.6.4 Removal from the tub

- Drain the bathwater
- The Bathlift Peter Pan® Long should be in the uppermost position for cleaning
- Rinse the Bathlift Peter Pan® Long with fresh, lukewarm water
- The suction force of the front suction cups is released by lifting the back of the device.
- Lift the Bathlift Peter Pan® Long out of the tub by the seat

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

3.6.5 Behaviour in case of a fault

The following information helps you with troubleshooting:

Fault	Possible cause	Fault resolution
Bathlift Peter Pan® Long moves with difficulty or not at all	Foreign body is blocking the mechanism	Remove foreign body
Spindle or spindle nut doesn't move freely	Foreign body is blocking the mechanism	Remove foreign body
Doesn't stand securely without slipping	Rubber feet severely soiled or damaged	Clean rubber feet or replace if necessary
Sprocket doesn't turn	Safety pin is broken	Repair by qualified personnel is required

If you are unable to rectify the fault despite this guide, please contact your responsible medical supply store or call our service number: **+49 (0) 9868 9339-0**.

3.7 Approved accessories, combination with other products

Petermann Bathlift Peter Pan® long is part of a comprehensive aid product range, which is effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

4 Product marking / CE

The Petermann Bathlift Peter Pan® long is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

5 Cleaning, care and disinfection

The Petermann Bathlift Peter Pan® long is made from rust-proof components and therefore doesn't require any special maintenance. A visual inspection for defects is required before each use. The device may not be used if there are any noticeable defects. Therefore, please contact your responsible medical supply store or call our service number.

For sanitary reasons, please clean the device on a regular basis. You can use common liquid household cleaner. After cleaning, all moving parts such as the threaded rod, gears, etc. can be coated with an oil-free lubricant (Vaseline) using a brush. The cover is machine washable up to 60°C.

If you do not wish to perform the inspection, cleaning and disinfection of your raising aid yourself, you can send your device to our shop.

If you don't use the Bathlift Peter Pan® Long for an extended period, store the Bathlift Peter Pan® Long in the uppermost position so that the spring mechanism is unloaded.

6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
42322-L	Drive mechanism: <ul style="list-style-type: none"> ➤ - high grade stainless steel and spring steel Seat: <ul style="list-style-type: none"> ➤ Polypropylene Back: <ul style="list-style-type: none"> ➤ Polypropylene Cover: <ul style="list-style-type: none"> ➤ Sympatex Polyester carrier material Handle: <ul style="list-style-type: none"> ➤ high grade stainless steel 1.4301 Handle box: <ul style="list-style-type: none"> ➤ plastic white Frame: <ul style="list-style-type: none"> high grade stainless steel 1.4301 	White	135 kg

Item no.	Description	Dimensions in mm	Weight	UDI
42223-L	Batlift Peter Pan® Long		16 kg basic device	(01)04250355343222(10) x
	Lying area height above:	420 mm on load		
	Lying area height below:	90 mm on load		
	Lift height	330 mm		
	Lying area	1480 x 700 mm incl. back (480 mm und 2 x side flap 146 mm)		
	Back		1 kg	

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

7 Warranty

- Petermann warrants that the Petermann Bathlift Peter Pan® Long is free from defects in material and workmanship.
- This warranty shall remain in effect for the following term **7 years**

Cover and suction feet are wear items and excluded from warranty

Please note that the warranty does **not** apply to:

- Repairs which were not performed by Petermann or authorised Petermann dealers
- Damage to or failure of the bathlift Peter-Pan Long which were caused by improper use
- Discoloration of covers or suction cups caused by the use of medical bath essences e.g.

8 Disposal information

There are no specific disposal regulations for Bathlift Peter Pan® long, it can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de