# HUMAN CARE

Your Life. Your Way.

User Manual/Bruksanvisning/  
Benutzerhandbuch/Gebruikershandleiding

FlexiRail, art. nrs: 90900-250, 90900-300,  
90900-350, 90900-400

# Content

---

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## Important!

Always read the user manual before using your product and keep this manual for future use.

***“Caution!” triangles are used to notify of situations that demand extra care and attention.***

## Compliance and standards

### Compliances and Standards

---

Human Care is an ISO 13485:2016 certified Swedish medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product is CE marked in accordance with EU Regulation 2017/745 (MDR), as class I medical device.

The CE mark is on the product.

This product has been tested in accordance with ISO 10535.

## **Before use, ensure that:**

- You have read and understood the instruction guides for the FlexiRail gantry.
- You have read and understood the instruction guides for the ceiling lift and lifting accessories.
- Personnel using the equipment have received appropriate instruction and training.
- Foot, leg kit and rail profiles are assembled correctly.
- The rail roller is in place and rolls freely in the rail.
- All screws are tightened and the washers are in correct position.
- Ensure gantry is placed on a level floor.
- Ensure that the height on both sides are set to the same height.
- Ensure that locking pin is secure / locked.
- If used with castors, ensure that all four castors are locked before performing a lift.
- You have the most recent version of the manual which is available for downloading from our website at [www.humancaregroup.com](http://www.humancaregroup.com).

***Warning! Product modifications is strictly forbidden without approval from manufacturer.***

***The FlexiRail should only be used indoors.***

***Lifting and transporting a person always involves a risk. It is important to read the instructions for both the lift and the lifting equipment. Always ensure that the equipments that are being used together with the lift is suited for the purpose. As caregiver you are responsible for the safety of the client and need to be aware of the client's ability to manage the lifting situation.***

## Symbols on the unit and product label

### Product labels

The product label contains information concerning the equipment. The label is placed on the leg.

The freestanding frame is tested by an accredited testing institute and comply with the requirements of directives for medical technical Class I products.

 **Before usage, ensure that the correct part number (250, 300, 350, or 400) is marked on the label. If not, mark it correctly with a permanent marker pen.**

	CE marking
	Date of manufacture
	Manufacturer
	Max. load
	Instructions: Read the instructions and make sure you understand them before using this product
	Article number/reference number
	Serial number
	Medical Device
EAN	EAN-number
	01) EAN/GS1/GTIN (11) manufacturing date (21) serial number

**HUMAN CARE**  
Your Life. Your Way.
**FlexiRail**
  

<table style="width: 100%; border-collapse: collapse;"> <tr> <td style="border: 1px solid black; padding: 2px;">REF</td> <td style="padding: 2px;"> <input type="checkbox"/> 90900-250  <input type="checkbox"/> 90900-300  <input type="checkbox"/> 90900-350  <input type="checkbox"/> 90900-400                 </td> </tr> <tr> <td style="border: 1px solid black; padding: 2px;">SN</td> <td style="padding: 2px;">123456789012</td> </tr> </table> <p style="font-size: 8px; margin-top: 5px;">Human Care HC AB Årstabergsvägen 21B 117 43 Stockholm, Sweden Info@humancaregroup.com</p>	REF	<input type="checkbox"/> 90900-250 <input type="checkbox"/> 90900-300 <input type="checkbox"/> 90900-350 <input type="checkbox"/> 90900-400	SN	123456789012	<div style="text-align: center;">  <b>300kg / 661lbs</b> </div> <div style="text-align: center; margin-top: 10px;">  <b>YYYY-MM</b> </div> <div style="text-align: center; margin-top: 10px;">  </div> <div style="text-align: center; margin-top: 10px;">  </div> <div style="text-align: center; margin-top: 10px;">  </div>
REF	<input type="checkbox"/> 90900-250 <input type="checkbox"/> 90900-300 <input type="checkbox"/> 90900-350 <input type="checkbox"/> 90900-400				
SN	123456789012				

www.humancaregroup.com

UDI	
(01)7331599724569(11)yyyymm(21)123456789012	

Label 94340

## Product Description / Intended Use

ENG

FlexiRail is a freestanding gantry that has the possibility of adjusting width and height within the ranges 250cm, 300cm, 350cm and 400cm. The FlexiRail is intended to be used together with a ceiling lift that is not suspended from a building or structure. It is suitable for home care, long term care, institutional care.

### **Article numbers FlexiRail in different widths:**

90900-250 (250cm rail)

90900-300 (300cm rail)

90900-350 (350cm rail)

90900-400 (400cm rail)

***Note! The lift and rail roller are ordered separately.***

***Do NOT move, transfer, or adjust the freestanding frame when being used with a client. If there are any doubts, please contact the supplier.***


## FlexiRail Dimensions

Product name / Article number	Height cm/inch	Length cm/inch	Width, feet cm/inch	SWL kg/lb	Product weight kg/lb
FlexiRail (250cm) / 90900-250	180 - 270cm 70.9" - 106.3"	265 - 325cm 104.3" - 128"	120cm 47.2"	300kg / 661lb	47kg / 103.6lb
FlexiRail (300cm)/ 90900-300	180 - 270cm 70.9" - 106.3"	315 - 375cm 124" - 147.6"	120cm 47.2"	300kg / 661lb	49.2 kg / 108.5lb
FlexiRail (350cm) / 90900-350	180 - 270cm 70.9" - 106.3"	365 - 425cm 143.7" - 167.3"	120cm 47.2"	300kg / 661lb	51.4kg / 113.3lb
FlexiRail (400cm) / 90900-400	180 - 270cm 70.9" - 106.3"	415 - 475cm 163.4" - 187"	120cm 47.2"	300kg / 661lb	53.6kg / 125lb

## Product Weight

**Side profiles:** 36kg / 79.4lb (18kg / 39.7lb per piece)

**Rails:** 1 meter / 39.4" rail = 4.4kg / 9.7lb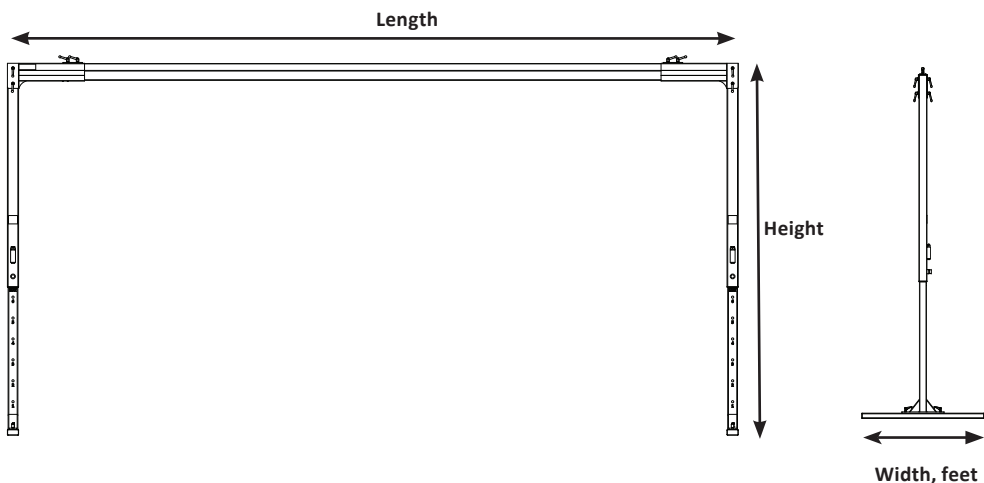


## Material

Rail, side profiles, feet: Powder coated extruded aluminium (White)

Corner assembly: powder coated steel (White)

## Included parts with product

#	Part	Article number	Amount
1	Rail 250cm, 300cm, 350cm, 400cm	72480, 7124, 72481, 71242	1
	<b>Leg kit</b>		
2	Side assemblies	94025	2pcs
3	Corner pieces	94045	2pcs
4	Feet	94052	2pcs
5	M10 levers and washers	94091, 94092	8pcs
6	M10 lever nuts and spring washers	94254, 94255	4pcs
7	M8 levers and washers	94088, 94259	4pcs
8	Rail connection plate	94055	2pcs
9	Connection plate	94046	2pcs
10	End stop	94245	2pcs
11	Allen Key 4mm		1pcs
12	Rubber mat	94258	4pcs

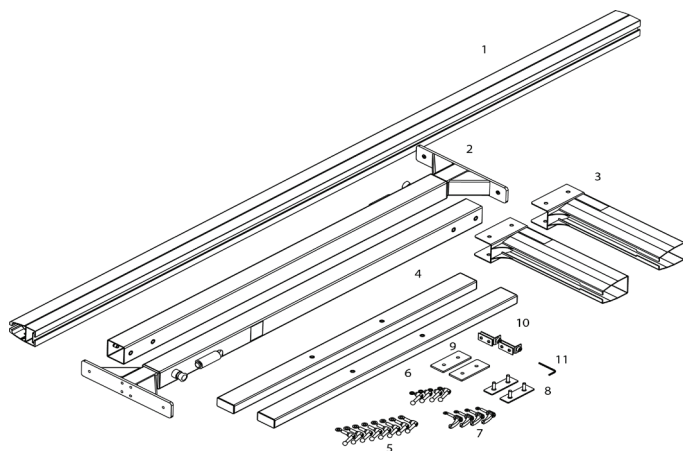**Note! We recommend that the following steps for assembly are done by 2 persons.**



If Flexirail is used with castors:

**Assembly instructions for Castor kit, 4 pcs,  
art. nr. 50872**

Tools required: 13mm wrench.


Attach the castors to the ends of each foot using  
a 13mm wrench.

Ensure that the wheels are locked when  
performing the rest of the assembly, instructions  
on following page.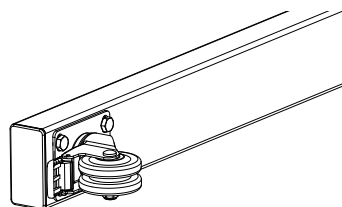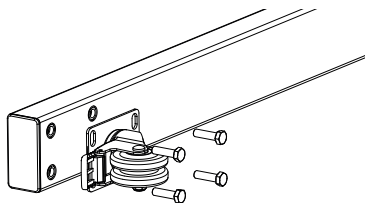

1.



Place the rail roller in the rail.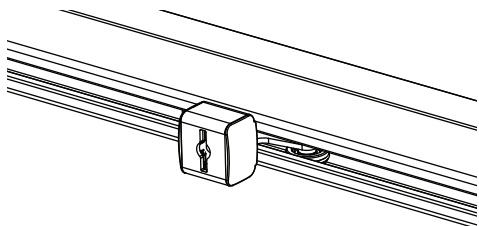

2.

Assemble the Connection plate, Rail connection plate, 2xM10 Lever nuts and 2x spring washers to the corner piece.  
(Leave a gap of ca 6mm between the rail connection plate and the corner piece)



Do this on both sides.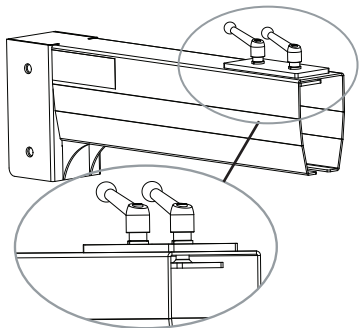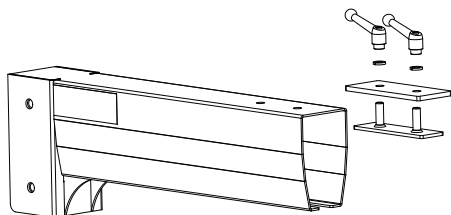

3.

Attach the rail to the corner assembly by lining up the rail connection plate with the slot in the rail and inserting the rail into the corner piece.



Attach the other corner piece to the other side of the rail.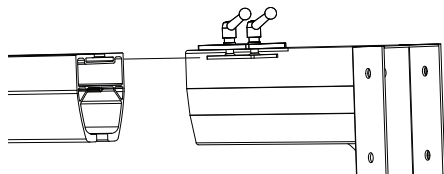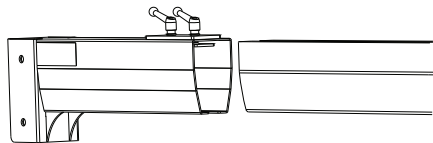

4.


Place the end stop in the corner piece and assemble it to the end of the rail with the rubber bumper towards the rail roller. Tighten the screws using the 4mm allen key.

Repeat this for the other side.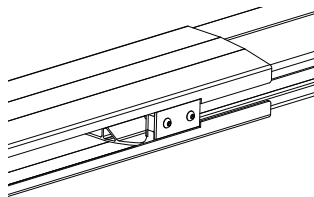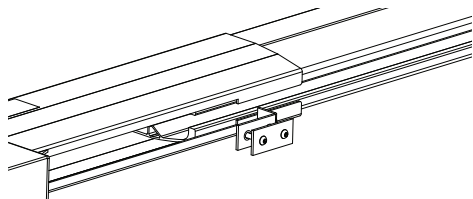


5.

Ensure that the assembled pole is mounted on the foot as shown before the screws and nuts are tightened.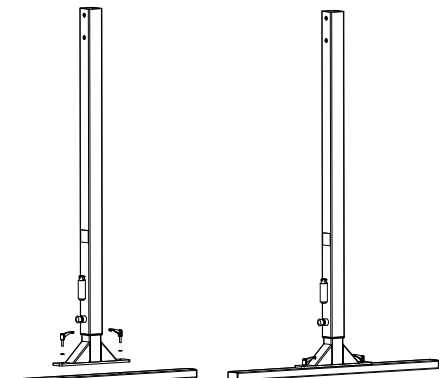

7.


Lift up the other side of the rail and place the corner piece on top of the other side assembly. Use 4xM10 levers and 4 washers to secure the assembly.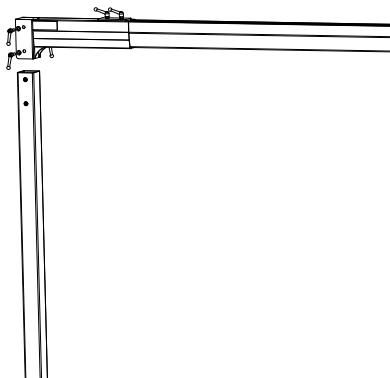

6.

Lift up one side of the rail and place the corner piece on top of one of the side assemblies.


Use 4xM10 Levers and 4 washers to secure the assembly.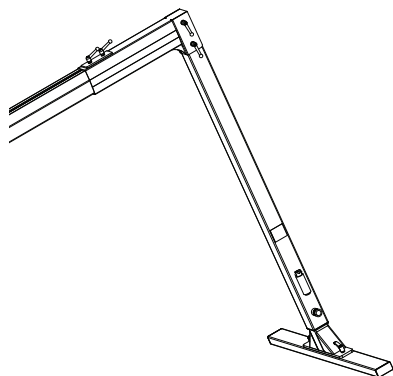

## Width adjustment

The width of the gantry can be adjusted 30cm on each side.

To adjust the width on one side, loosen the lever nuts on top of the corner and slide the side assembly. After the adjustment is done, make sure to tighten the lever nuts again.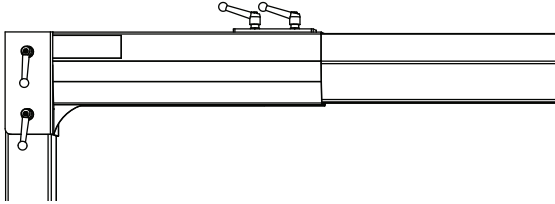

## Height adjustment

The height of the gantry can be adjusted in six steps from 180cm to 270cm.


To adjust the height:

Pull up one side of the gantry using the handle to disengage the locking mechanism (You can step on the foot on to keep it stable).

Pull the locking pin out.

Adjust the height up or down, one step at a time.

Adjust the height of the other side to the same level.

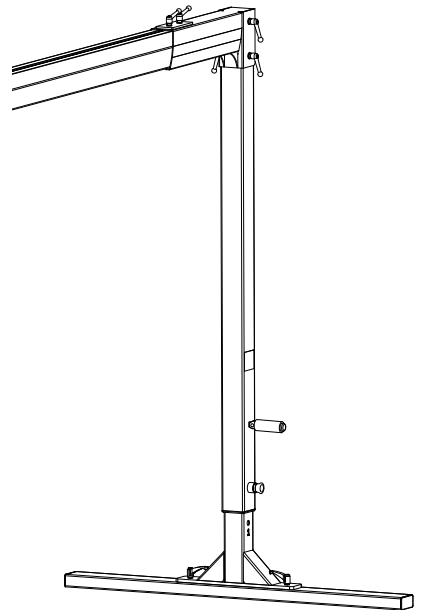# Placing the FlexiRail correctly

ENG

The freestanding gantry must always be positioned so that the client's center of gravity is directly under the rail. If the freestanding gantry is positioned incorrectly, stability will be compromised.

**Do not place the FlexiRail gantry at an inclined angle of more than 5° in any direction.**

**Do NOT move or transfer the FlexiRail gantry when being used with a client.**

Correct position

Incorrect position

**Photo: FlexiRail and bed seen from above**

# Troubleshooting

ENG

The rail roller does not run smoothly.

- 1) Check if there is something jammed in the rail.
- 2) Check the rail roller wheels for damage.
- 3) Check that the nylon washer on the axis of the loop on the rail roller is correctly positioned under the rail.

## Care of the freestanding frame

- When necessary the FlexiRail gantry can be cleaned with a damp cloth and dishwashing liquid. Do NOT use strong detergents. Dry with a dry cloth after.
- Never flush the freestanding frame with water or other liquids.

## Service agreements

Human Care invites you to sign a service agreement for regular maintenance and testing.

## Storage

The climatic conditions should be as follows, an ambient temperature from 0 °C to 50 °C, relative humidity from 30 % to 80 % and air pressure from 790 hPa to 1060 hPa.

## Recycling

Once the product has reached the end of its technical life, it must be disposed of in constituent parts according to local authorities. Metal parts must be treated as metal waste for optimum recycling. Plastic parts can be sorted as residual waste.

## Annual inspection

The freestanding frame must be inspected at least once a year by Human Care authorized personnel. Pay particular attention to parts that show wear. Service and maintenance are to be performed by Human Care authorized personnel using original spare parts.

# Accessories

For more information on all accessories available such as rail rollers and ceiling lifts see [www.humancaregroup.com](http://www.humancaregroup.com).

If the safe working load for any of the accessories, e.g. slings, lift bars and any other accessories is NOT the same as for the lift, then the lowest safe working load given for any one product is the one that shall apply. Always check the label on the lift and lifting accessories or contact Human Care if you have any questions or if anything is unclear. Note: accessories are not included with the product.

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Human Care USA Inc.