Kenmak[®]

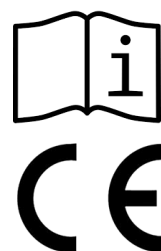
part of the Human Care Group

producing technology for health

User manual / Bruksanvisning

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Art nr: K048

Content

- 1- Introduction**
- 2- Intended Use**
- 3- Warnings**
- 4- Assembly Instructions**
- 5- Technical Details**
- 6- Accessories**
- 7- Product Labelling**
- 8- Environment, Transport and Storage**
- 9- Maintenance and service**
- 10- Warranty / Life age**

1- Introduction

Thank you for choosing your hospital furniture from Kenmak. Every single Kenmak Hospital Furniture product is being produced under high quality standards bearing in mind how to minimize the risk of environmentally reverse effect. Kenmak Hospital Furniture's operation are certified by globally recognized authorities according to ISO 9001, ISO 13485 and ISO 14001 standards.

Kenmak Hospital Furniture products are manufactured in Turkey in line with aforementioned quality standards and are tested before dispatching the factory by our Quality Team.

Any incident involving the product should be reported to the manufacturer and a relevant regulatory body.

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www.kenmak.com.tr

2- Intended Use

The Freestanding IV pole, K048, is intended to hang bags with intravenous fluids (IV) in professional care environments.

3- Warnings

Using the equipment for purposes other than those previously described is forbidden. The equipment must only be used by properly trained person.

Before starting to use the IV pole, the operator must be perfectly aware of how it works and of all technical-functional features. Such awareness can be acquired by carefully reading this manual.

If it is possible, the nursing staff should properly inform patients on how the IV pole works. After such information has been provided, if it is possible, the patient should be asked to "trial run" the IV pole.

Removing any safety or protection devices on the appliance is strictly forbidden; periodically, at weekly intervals, check the integrity of such devices and make sure they are working properly.

Before carrying out any maintenance or repair jobs on the equipment, the operator should have carefully read and understood the technical information provided in this manual. All maintenance and repair jobs must be done by skilled personnel.

Never position the equipment close to heat sources.

Position and secure the equipment in a sIV-pole manner on a horizontal floor. The area around the IV pole must be kept clean, in order and devoid of objects that could restrict free movement. The instructions, accident-prevention rules and warnings contained in this manual must always be followed.

4- Assembly Instructions

Number	Description	Qty.
1	K048	1

Parts required for installation:

Preinstalled star shape casting foot.

Preinstalled IV Pole body

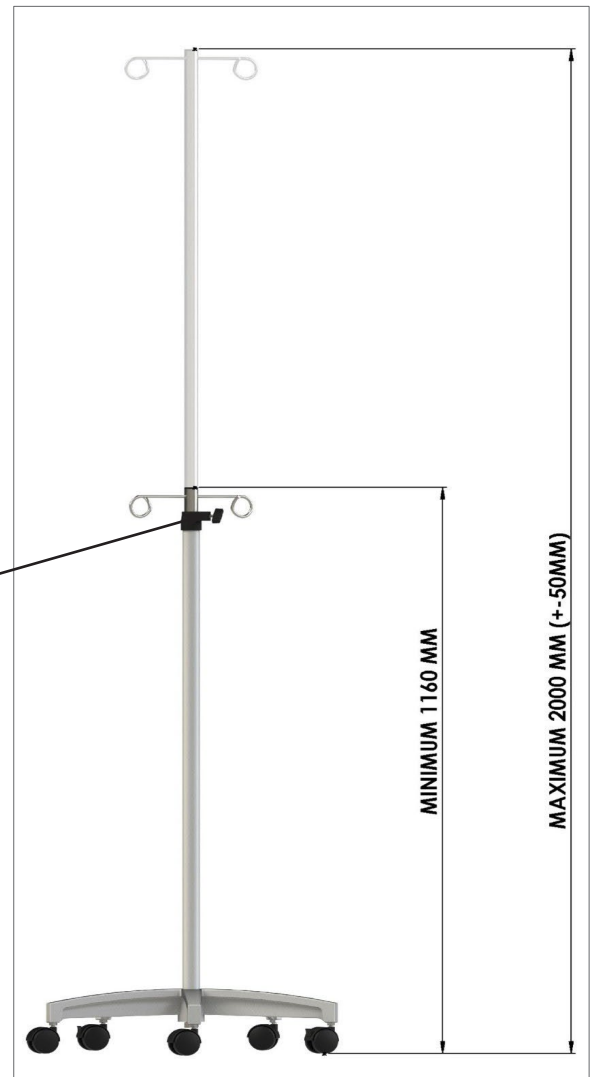
Allen key Washer Bolt

As seen in the upper picture, the upper body and the lower body are combined and fixed with a bolt from the bottom with the help of an allen key.

The height of the IV pole is fixed with the help of the handpiece.

- Lower leg is completely cast aluminum and powder oven painted.
- The IV pole has 50mm plastic wheels, 2 with brakes and 3 without brakes.
- The upper IV pole body is chrome plated.
- Height adjustment is practical and easy with the butterfly grip.
- It is very easy to disassemble. By removing a single bolt under the IV pole, the lower leg and the body are separated from each other.

5- Technical Details

Specifications (cm / inch - kg/lbs)	
Height (Min-Max)	1160-2000
Width	566
Safe Working Load	7 kg/ 15 lbs

6- Accessories

Please visit www.kenmak.com.tr for additional accessories.

Kenmak®

part of the Human Care Group

producing technology for health

7- Product Labelling

 part of the Human Care Group					
MANUFACTURER: KENMAK HASTANE MALZEMELERİ VE ELEKTROSTATİK SAN. TIC. A.Ş. Tel: +90 232 274 17 17 (pbx) Fax: +90 232 274 67 77 www.kenmak.com.tr kenmak@kenmak.com.tr Sarıç Yolu No:23 Gaziemir-İZMİR-TURKEY (Made in Turkey)		Kenmak Hospital Furnitures & Equipments Sarıç Yolu No:23 Gaziemir-İZMİR-TURKEY 			
Model Name : K048		Production Date : _____ Serial Number : Weight : Dimensions : 60 x 60 x116 (h) _____			
Medical Device Class: I		Human Care HB AB Årstaängsvägen 21B 117 43, Stockholm / SWEDEN T: +46 8 510 132 00; F: +46 8 665 35 10 info.se@humancaregroup.com			
					

8- Environment, Transport and Storage

Environment

Kenmak is certified in accordance with ISO 14001 and work with continuous improvement to reduce total environmental impact. A primary focus is logistics where we have reduced our total environmental footprint.

- Because the top IV-pole, height adjustment handle and tilt adjustment handle are made of plastic, it can be reused in plastic recycling.
- Metal body parts as upper IV-pole lower connection piece, middle body inner profile, middle body outer profile and lower tube profile can be sent to metal recycling.
- The wheels can be sent for separation first, and then the metal and rubber parts can be sent to the relevant recycling separately.
- Packaging materials should be sent to paper recycling and nylon recycling.

Our products are recyclable. Help our environment and hand in the product for recycling. Follow local instructions for recycling.

9- Maintenance and service

Inspection, care and maintenance check list

User inspections of this accessory should be carried out every 6 months

Check: See product description, accessories and all functions	<ul style="list-style-type: none"> • Check for Damage/Cleanliness • Confirm Secure • Perform Adjustment/Cleaning 	OK	Faults: Action Cleaning Parts to Order
VISUAL CHECK of the Components			
Nuts/Bolts, Screws/Pins, Hinges/ Mounts/Bushes-component fixing points	Wear/Damage, Tighten & Secure Clean & Free		
Painted steel components	Intact, Clean & Securely Fitted		
PERFORMANCE CHECK			
Accessories	Confirm Full Operation		
Inspector's Name:	Inspector's Signature:		
Inspection result:	Date:		

If any problem persists, please call the customer service in your country.

10- Warranty / Lifeage

The product's expected lifetime is five (5) years, if the product is used as intended and maintained according to the manufacturer's instructions, depending on the intensity of use and maximum load applied during use. If the product label is no longer legible, the product should be discarded

Please contact us for any questions:

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Human Care AB Årstaängsvägen 21B, 117 43 Stockholm/SVERIGE