HUMAN CARE

Your Life. Your Way.

Assembly Instructions / Instructions d'assemblage

/ Montageanleitung/ Montage instructies

Slow Down Brake for neXus 3 and neXus oNe

Art.nr 93122

Assembly Instructions

Art. nr: 93122

Assembly instructions

These assembly instructions explain how to mount the Slow Down Brake on the neXus rollators rear wheels. The Slow Down Brake is delivered in pairs, a brake for the **right rear wheel** and a brake for the **left rear wheel**.

Scan the QR-code with your mobile phone to see the assembly instruction video.

1, Ensure the small wheel cassette is inserted in the slowdown brake during the assembly.

2, Take the slow down brake and loosen it by unscrewing the knob, so the small wheel cassette goes deeper into the slowdown brake.


3, Lay the rollator down on its back frame, with the front wheels facing upward into the air. **The rear wheels** should be fixed on the floor and parallel to each other.


4, Place the slow down brake diagonally against the rear wheel.

5, The clips on the slowdown brake should be inserted diagonal into the fork frame.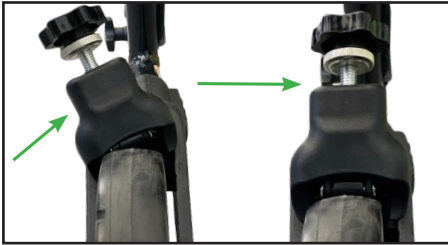

6, When the clips are inside the frame, twist the slow down brake into the fork frame, it should **click** into place.

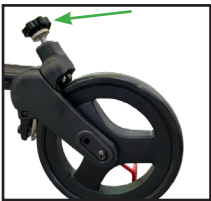7. Once the slow down brake is in place, push the knob forward and listen for another **click**.

8. Repeat the process for the second wheel.

9. Stand the rollator up and push the slowdown brakes towards the floor to ensure that they are securely attached.

Disassembly instructions

1. First disassemble the back fork by removing the four screws shown in picture.
2. Remove the slow down brake from the fork.
3. Re-assemble the fork by mounting the four screws again.

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